**MOBICLINIC**

Spanish Leading Company

Since 1985

Seville, SPAIN

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Since 1985

Seville, SPAIN

**User manual  
Electric wheelchair  
Troya  
Troya PLUS  
Troya PLUS ULTRA**

EN- Please read instructions before use.

User manual in English

EN

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Last revision: June 11th 2022

# INDEX

1. Packing list of standard wheelchair	2
2. Main structure	2
3. Open or fold wheelchair	3
4. Electrical principle diagram	4
5. Install joystick and connect	5
6. Install batteries and remove	6
7. Drive and Controller; turn on or turn off	6
7.1. Function instructions of standard brushless controller	8
8. More functions	9
8.1. Flip-up armrest	9
8.2. Foldable footrest	9
8.3. Height adjustable backrest	9
9. Charging lithium batteries	10
10. Technical characteristics	11
11. Malfunction detection	12

Before using the Troya series electric wheelchair, please read this user manual carefully and save it for future use.

**Warning**

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

**Indications**

This product is intended for disabled or elderly people who have difficulty moving.


**Contraindications**

Not recommended for people who are too weak to sit up. Person who has difficulty operating the wheelchair due to various disabilities.

# 1. PACKING LIST OF STANDARD WHEELCHAIR

# 2. MAIN STRUCTURE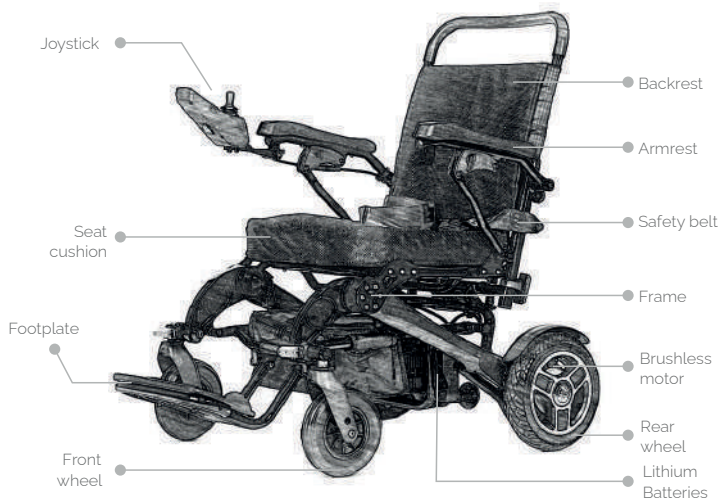


### 3. OPEN OR FOLD WHEELCHAIR


**METHOD 1** To open the wheelchair:

**METHOD 2** To open the wheelchair:

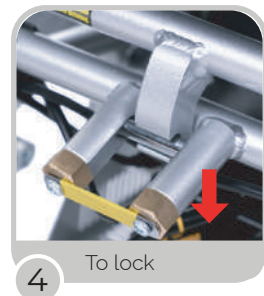**Fold the wheelchair:**
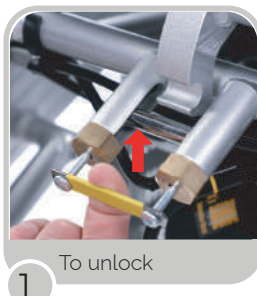
# 4. ELECTRICAL PRINCIPLE DIAGRAM

Depending on the controller model of your wheelchair:


LH-C Controller


SY-C20 Controller


LH-C CONTROLLER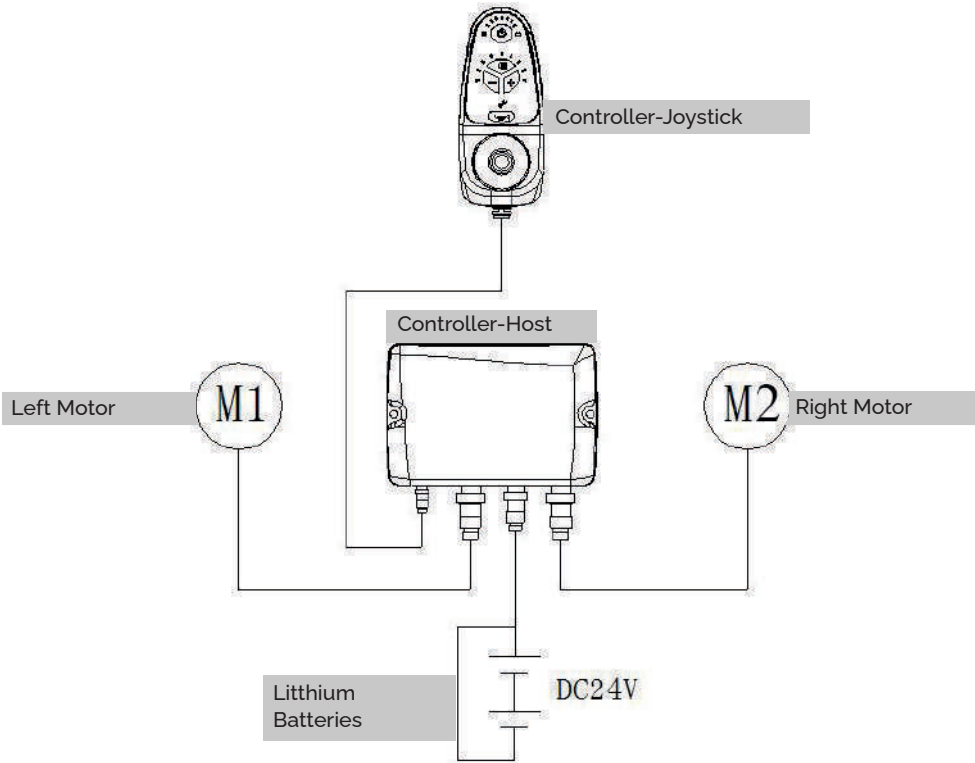

SY-C20 CONTROLLER

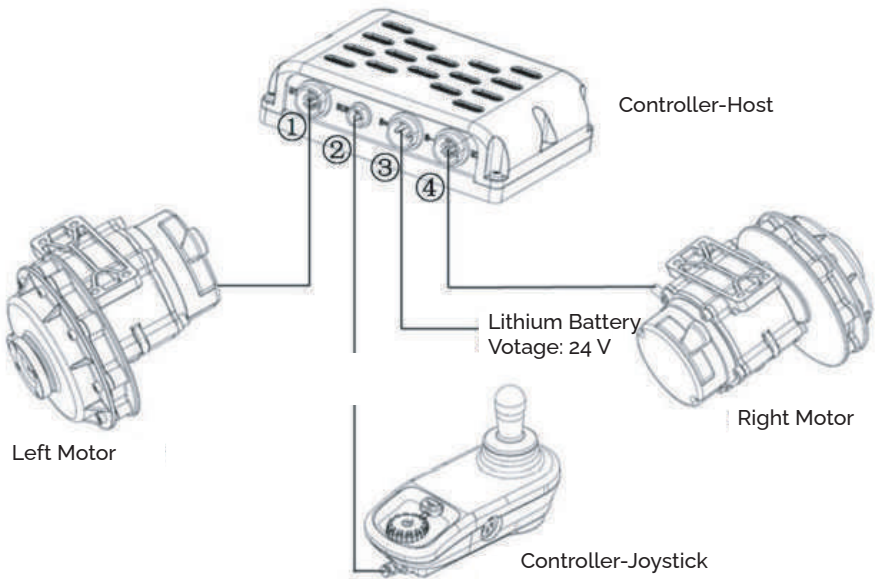
## 5. INSTALL JOYSTICK AND CONNECT
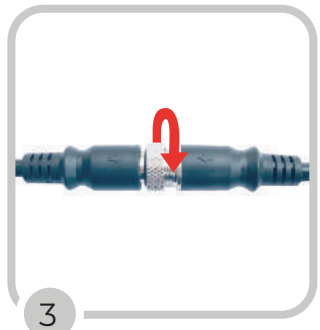
## 6. INSTALL BATTERIES AND REMOVE


### INSTALL

Lithium batteries: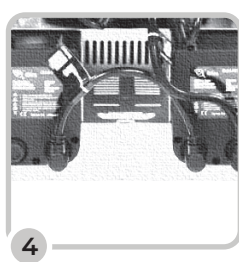

### REMOVE

Lithium batteries:

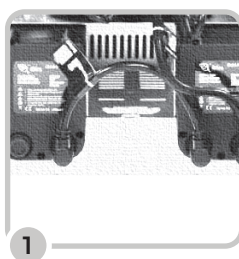## 7. DRIVE AND CONTROLLER; TURN ON OR TURN OFF

Switch to Driving Mode

When lever is in the M (Manual) position, the electric wheelchair can be moved without power. When lever is in the D (Drive) position, the electric wheelchair can be driven

Standard controller

### LH-C CONTROLLER

Switch between Manual and Drive

**LH-C** Brushless Controller

**LH-D** Brushless Controller (smaller)

Square wave  
Brushless motors & controllers

### SY-C20 CONTROLLER

Switch between Manual and Drive

**SY-C10** Brushless controller LCD

**SY-C20** Brushless Controller LCD

Sine wave LCD  
Brushless motors & controllers

# 7.1. FUNCTION INSTRUCTIONS OF STANDARD BRUSHLESS CONTROLLER

Battery energy indicator light

Speed and break-downs indicator light

Maximum speed setting by + and -

Horn

Power button

Engineering mode button and indicator light

Rocker (or Joystick)

Charging and communication port on the bottom of the unit

LH-C

Real time driving speed display

Maximum speed display

Maximum speed setting by + and -

Charging and communication port on the bottom of the unit

Battery power display

Power button

Horn

Rocker (or Joystick)

SY-C20

## 8. MORE FUNCTIONS

### 8.1. FLIP-UP ARMREST

Unlock

Fold

Lock

### 8.2. FOLDABLE FOOTREST

### 8.3. HEIGHT ADJUSTABLE BACKREST

## 9. CHARGING LITHIUM BATTERIES

Charging time usually is 2-8 hours.

**METHOD 1** To charge by joystick:

1

1 Connect the charger

2

2 Connect the joystick

3

3. Connect the power socket

**METHOD 2** To charge by cable ( Directly charge for lithium batteries):

1

2

3

### Indicator light of Charger:

**Red light on:** Power on, disconnect the batteries.

**Orange light on:** Charging.

**Green light on:** Fully- Charged.

### CAUTIONS OF CHARGING:

1. Only for indoor use, don't expose to rain.
2. Please switch off the power supply before removing the charger from the batteries.
3. Do not get closed to explosive gases or sparks and put the charger in a well-ventilated area.
4. Use the charger only with 24V lithium batteries.
5. Charge the battery every 3 months if not in use for a long period of time.

## 10. TECHNICAL CHARACTERISTICS

PARAMETERS	
Motor	LH-C: 250W*2, DC 24V / SY-C20: 220W*2, DC 24V
Maximum Speed	8,5 km/h
Braking distance	≤ 1,0 m
Maximum slope capacity	15°
Turning radius	85 cm
Rear wheel	30,48 cm / Maximum preassure 2,5 bar
Front wheel	20,32 cm
Net weight (without battery)	24,8 kg
Maximum user weight	120 kg
Lithium battery	24V10Ah/pc
Net weight of the battery	1,7 kg/pc
Autonomy	LH-C: 17 km/1 battery-34 km/2 batteries-51 km/ 3 batteries SY-C20: 20 km/1 battery-40 km/2 batteries-60 km/3 batería
CD charger output	24V/2A
CA charger input	100 - 240V, 50/60Hz

MEASURES	
Seat width	450 mm
Seat depth	440 mm
Seat height	500 mm
Wide backrest	440 mm
Width of the backrest	470 mm
Distance from the seat to armrest	210 mm
Ground clearance	65 mm
Distance between axles	600 mm
Total lenght	1040 mm
Full width	640 mm
Overall height	910-1060 mm (height adjustable)
Measurements folded	640 x 340 x 740 mm (0,16m <sup>3</sup> )
Cardboard box size	730 x 420 x 840 mm (0,26m <sup>3</sup> )

# 11. MALFUNCTION DETECTION

## LH-C CONTROLLER

When the speed indicator light flashes on the controller, it indicates that the power wheelchair is in an abnormal condition. The meaning of the flashing status and the recommended elimination method are shown in the following table.

Speed indicator light Flashing status	Description of the fault and warning	Solution
 First light flashing	Failure in the left engine compartment	1. Check if the left engine compartment is damaged; 2. Check whether the left motor cable is connected or not;
 Second light flashing	Failure in the right engine compartment	1. Check if the right engine compartment is damaged; 2. Check if whether the right motor cable is connected or not;
 Third light flashing	Low battery level	Charge the battery (first step) and check the battery level again.
 Fourth light flashing	Left engine failure	Check the left motor and its connections
 Fifth light flashing	Right engine failure	Check the right motor and its connections.
 Sixth light flashing	Left brake failure	Check the left engine brake and its connections.
 Seventh light flashing	Right brake failure	Check the right engine brake and its connections.
 Eighth light flashing	Controller failure	90% probability, the malfunction is caused by the power tube of the control unit, it is suggested to replace the control unit or the main controller, or contact the after-sales service, please.
 Ninth light flashing	Joystick and controller connection failure	1. Check the connection of the joystick cable with the controller. 2. The controller is damaged.
 Tenth light flashing	Motor overload	Release the control bar to resume.

Speed indicator light Flashing status	Description of the fault and warning	Solution
 First and tenth light flashing	High battery voltage	Check the output voltage, it must not be higher than 35V.
 All lights flashing	The rocker is not in the center of the joystick	<ol style="list-style-type: none"> <li>1. Do not touch the rocker arm when its switched on;</li> <li>2. Switch off, shake the rocker, and try to bring it back to the centre, if the problem persists, please contact the after-sales service.</li> </ol>

## SY-C20 CONTROLLER

When the fault code appears on the controller display, it indicates that the power wheelchair is in an abnormal condition. The meaning of the display and the recommended clearance method are shown in the following table.

Fault code	State of the indicator	Sound of the alarm	Description of the fault and warning	Method of elimination
C1	Long flashing	Long time alarm	Communication failure	Check if the wiring harness is correctly installed and connected
C2	Long flashing	Long time alarm	The joystick is not in the initial position	Disconnect the power supply, release the joystick to the centre position, and then switch it on.
C3	Long flashing	Long time alarm	Control conflict (if your wheelchair has front controller and rear controllers)	Disconnect power to the front controllers and the rear controllers, and select one to turn one.
C4	Long flashing	Long time alarm	Braker lever has not been set to electric state	Check if the engine is in running condition
C5	Long flashing	Long time alarm	Overload warning (the first step)	Reduce the wheelchair load or release the joystick to the centre position, the wheelchair reset.
C5	Short flashing	Short time alarm	Overload warning (the second step)	
C5	Long flashing	Long time alarm	Overload warning (the third step)	
C6	Long flashing	Long time alarm	Ignition failure	Reset
C7	Long flashing	Long time alarm	Left and right wheel overvoltage warning	The joystick returns to the centre position until the overvoltage warnings is removed from the overvoltage warning.
C8	Long flashing	Long time alarm	Left and right wheel voltage warning	Please, charge the battery in good time.
CE	Long flashing	Long time alarm	Failure in system parameters	Restart or contact the sales service.
L1	Long flashing	Long time alarm	Current left-hand wheel overcurrent failure	
L2	Long flashing	Long time alarm	Excessive deflection of the current left-hand wheel	

Fault code	State of the indicator	Sound of the alarm	Description of the fault and warning	Method of elimination
L3	Long flashing	Long time flashing	Left wheel circuit overheating warning	Disconnect power, restart after drive wheel cools down.
L6	Long flashing	Long time alarm	Failure of the left wheel encoder	Restart or contact the sales services.
L7	Long flashing	Long time alarm	Left wheel brake failure	
L8	Long flashing	Long time alarm	Left wheel phase failure	
R1	Long flashing	Long time alarm	Overcurrent failure on the right wheel	
R2	Long flashing	Long time alarm	Excessive deflection of the current right-hand wheel	
R3	Long flashing	Long time alarm	Right wheel circuit overheating warning	Disconnect power, restart after drive wheels cools down.
R6	Long flashing	Long time alarm	Failure of the right wheel encoder	Restart or contact the sales services.
R7	Long flashing	Long time alarm	Right wheel brake failure	
R8	Long flashing	Long time alarm	Right wheel phase failure	

If the problems shown above cannot be solved, please do not disassemble the wheelchair by yourself. Please contact the after sales service.

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SUZHOU IDEAL MEDICAL CO LTD
SLUNGO Europe B.V. legal en la UE:
Imported by/Importado por: Grupo R. Queraltó S.A.