MOBICLINIC

Spanish Leading Company

Since 1985

Seville, SPAIN

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INSTRUCTION MANUAL

NOTICE

Notice to the user and/or patient: any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established.

INDICATIONS

The product is recommended for people with mobility difficulties, who can operate the device with one hand.

The wheelchair can be used indoors and outdoors. Consideration should be given to the environmental conditions and surroundings where it is to be used.

Max. weight	
Lyra	120 kg
Cenit	110 kg
Orión	120 kg

ASSEMBLY AND USE

1. To unfold the wheelchair, stand the wheelchair upright and press and hold both sides of the seat support until it opens fully (Figure 1). To fold, lift the seat cushion from the centre (Figure 2).

Figure 1

Figure 2

2. Attach the footrest (Figure 3). The height of the footrest can be adjusted (Figure 4). To remove it, simply pull it out.

Figure 3

Figure 4

3. Install the anti-tip wheel and fit the fixing bolt (Figure 5).

Figure 5

Figure 5.1

4. To install the batteries, with the wheelchair open, hook the pins on both sides of the wheelchair to the support rods (Figure 6). This will connect the batteries to both sides of the wheelchair (Figure 6.1).

Figure 6

Figure 6.1

5. Install the controller. To do so, insert the bracket into its corresponding location on the armrest, adjust the location of the controller and tighten the adjustment bolt. Connect the wires according to the picture (Figure 7). The one marked LEFT connects the left motor and the one marked RIGHT connects the right motor.

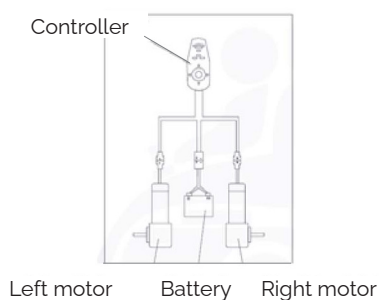
Figure 7

6. Attach the protective strap by hooking it to the footrest bars (Figure 8).

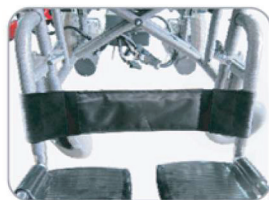
Figure 8

7. To start using the wheelchair, set the rear levers to Automatic Mode. If the levers are in Manual Mode, you will be able to move the wheelchair manually, but not use it as a power chair.

Automatic
ModeManual
Mode

Once you perform the installation steps described above, the wheelchair is ready for operation. To optimise battery performance, it is recommended to first fully charge the batteries before using the wheelchair for the first time.


CHARGE

The charging port of the powerchair is located on the bottom of the controller. The wheelchair uses two 12V batteries connected in series. To charge them, first connect the charger to the controller and then connect it to the power supply to start charging. When the battery is discharged, the indicator will be illuminated red; once the battery is full, it will turn green. If it is lit and flashing, the battery is charging.


1. Avoid using the wheelchair outdoors at low temperatures.
2. Stop using the wheelchair immediately if a low voltage alarm appears. To avoid excessive discharge and damage to the battery in normal use.
3. If the battery is overcharged, the battery life will be reduced.
4. If the wheelchair is not to be used for an extended period of time, allow the wheelchair to fully discharge and disconnect the controller plug from the battery.
5. Both batteries should be changed at the same time.
6. Do not disassemble the battery, as the acidic liquid it contains may be toxic to the human body.

CONTROLLER FUNCTIONS

ORION CONTROLLER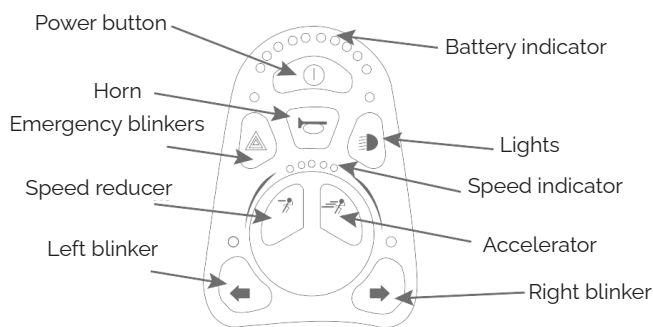

LYRA AND CENIT CONTROLLER

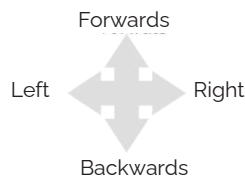LEVER DIRECTIONS

TECHNICAL SPECIFICATIONS

Maximum speed	6 km/h
Approximate range	20 km
Batteries	24V (2x12V)
Weight without batteries	42 kg
Charging time	6-8 h
Maximum tilt	8°
Operating environment	-5~40°C

CLEANING AND MAINTENANCE

To clean the product, use a slightly damp cloth, never wet, and be sure to dry it afterwards with another clean, dry cloth.

Do not use solvents, bleach, synthetic detergents, wax polishes, aerosols or other abrasives that may damage your product.

Store the product in a clean, dry place, away from rain, snow, ice, salt and water.

FAULT INDICATOR

If a fault should occur with your wheelchair when switching it on or during a journey, the protection system will automatically activate and send fault signals by means of a flashing light on the hand control. At this point, you should not use your wheelchair until the problem has been resolved.

Number of blinks	Fault	Solution
1	Faulty contact to charge the battery or batteries too low.	Check the cables to verify that the battery is properly charged. Charge the battery.
2	Faulty contact of the left motor circuit.	Check the connection and wiring between the control unit and the left motor.
3	Shortcircuit in the left motor.	The left side motor is short-circuited, consult a technician.
4	Bad contact of the right engine circuit.	Check the connection and wiring between the control unit and the right-hand motor.
5	Shortcircuit in the right motor.	The right side motor is short-circuited, consult a technician.
6	The controller has stopped working for safety reasons.	An external signal has interfered with the controller, check if the charger is connected. Do not use the controller while the battery is charging.
7	The steering lever of the control is faulty.	Move the lever to the neutral position and reset the control.
8	The control has an internal fault.	Consult with a technician.
9	The right and left brakes or their circuit is faulty.	There is a short circuit or damage to the electromagnetic brake circuits. Consult a technician.
10	Battery overcharge.	Check the battery, its connections and wiring.

PRECAUTIONS

- 1.** The electric wheelchair is suitable for indoor and outdoor use.
- 2.** Do not use on bumpy roads or climb steps frequently.
- 3.** It is not suitable to climb long slopes for long periods of time as this can cause the engine to overload and the battery to drain quickly.
- 4.** If you need to stop the wheelchair, reset the control lever and it will stop automatically.
- 5.** Do not disassemble the battery under any circumstances.
- 6.** If you need to fold the wheelchair, the battery must be disconnected and removed.
- 7.** Caution. Flammable material, avoid exposure to heat sources such as flames or sparks. Avoid transporting batteries close to flammable or combustible objects.
- 8.** Should you need to brake in an emergency situation, pull the steering lever in reverse and the wheelchair will stop instantly thanks to the system's electromagnetic brake.
- 9.** Keep tools and other metal objects away from the edges of the battery, as short circuits or electric shocks may occur in case of contact and cause injury.
- 11.** Extreme caution must be exercised when driving in reverse. The drive wheels could hit an obstacle and cause the wheelchair to lose control and tip over.

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GUANGDONG KAIYANG MEDICAL TECHNOLOGY GROUP CO., LTD
SUNGO Europe B.V.